


# ORTHOPUS ARM BRACE

## QUICKCHART




The position of both the **arm shell and elbow support is vital** to ensuring a good operation of the ORTHOPUS device

### INCORRECT POSITIONING

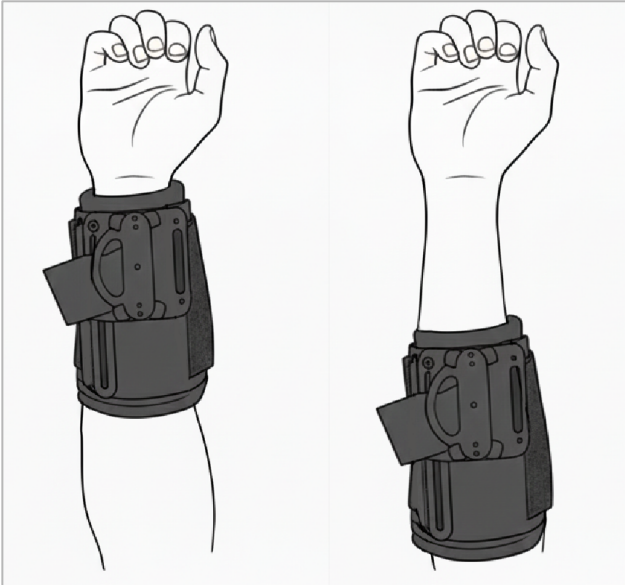


⊖ Arm shell **too close**  
**of the wrist or the elbow**

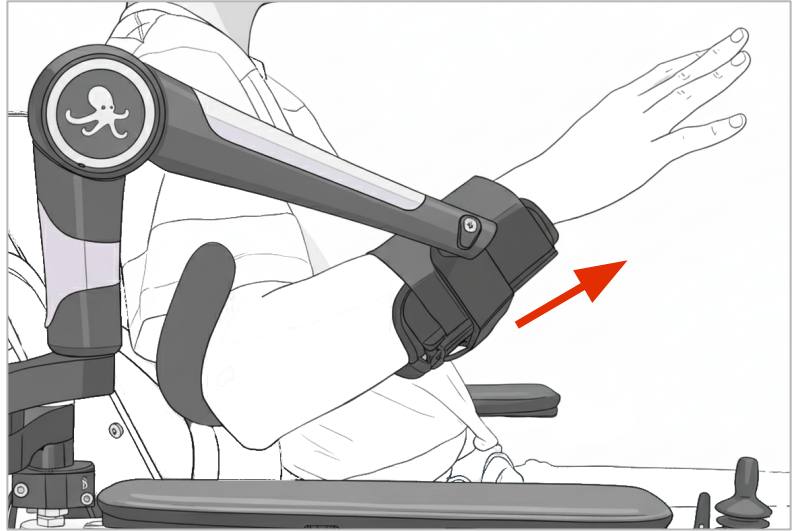


⊖ Arm **not aligned** with the arm rest.  
It should be aligned during setup.

### CORRECT POSITIONING

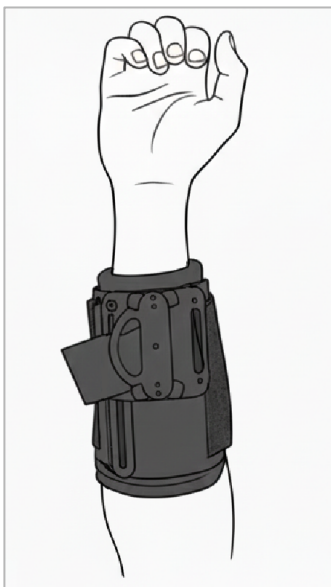


✓ Arm shell **well**  
**positioning**

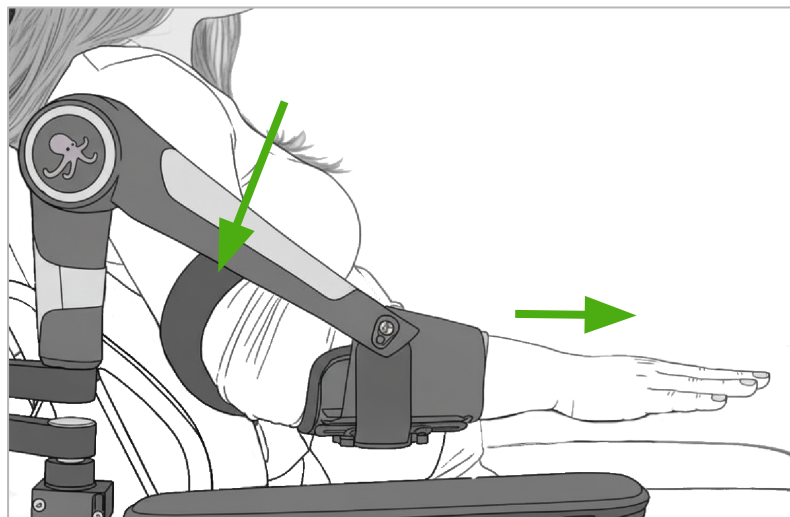


✓ Arm returns to a **horizontal resting**  
**position** after movement

✓ Elbow support **twisted upwards**  
to ensure a snug fit

## HOW TO FIX THE ARM SHELL & ELBOW SUPPORT

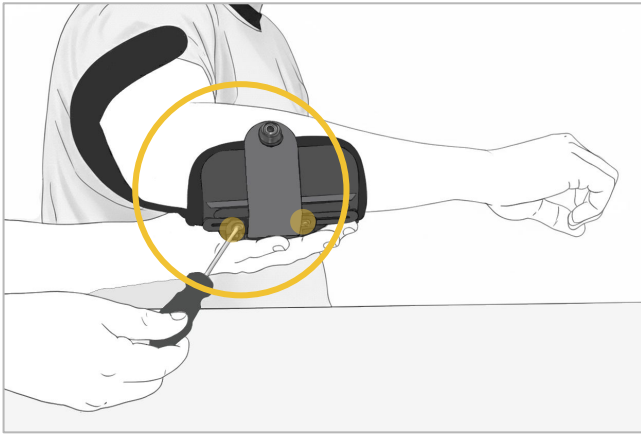


**2 outer screws**  
→ to fix the arm shell position

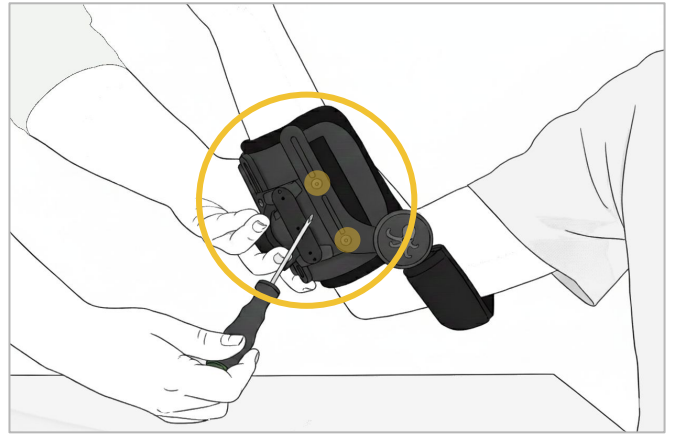


**2 inner screws**  
→ to fix the elbow support position

## HOW TO ADJUST THE ELBOW SUPPORT

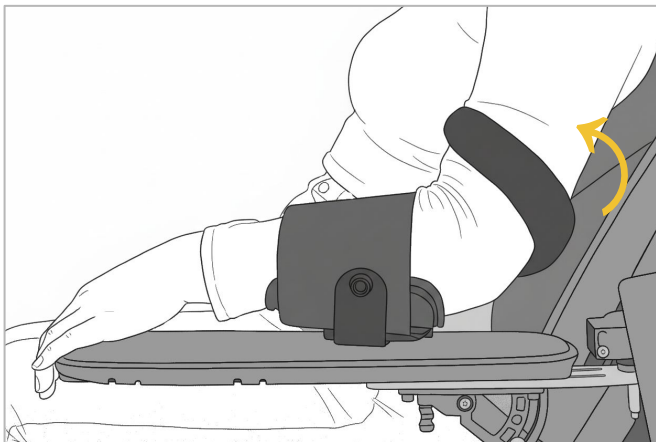

Twist the elbow support to push it **upwards**. The flat part must be **in full contact** with the arm

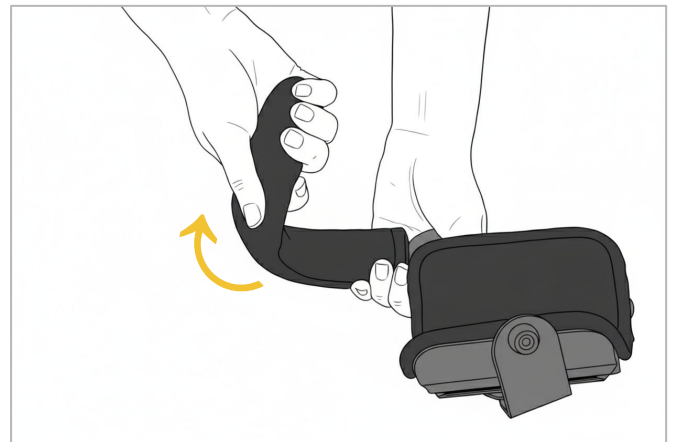

Feel free to remove the user's arm from the arm brace to do this more easily



Find **all the detailed information about the ORTHOPUS arm brace (4 sizes, adjustments)** in the Instructions For Use in the ORTHOPUS packaging box, or at [orthopus.com/documentation](https://orthopus.com/documentation)



Scan this QR code to watch our video **TUTORIAL - Installation and adjustment of the arm brace** (7min)



ORTHOPUS - [orthopus.com](https://orthopus.com) - [usercare@orthopus.com](mailto:usercare@orthopus.com)