


ORTHOPUS SUPPORTER

QUICKCHART

How-To
videos



REAR BUTTON' COLORS

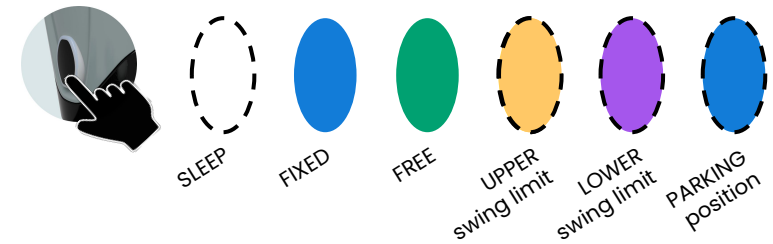

FIXED MODE = BLUE

Locks in the height chosen

—
down



+
up

SETTING SWING LIMITS



**1 LONG
CLICK**

Set the **upper**
swing limit



Set the **lower**
swing limit



Back in
FIXED mode



Position the device where you would like
to set the limit with the control pad


To **SET**, **1 LONG CLICK**
Long vibration = **OK**

To skip **WITHOUT** setting, **1 SHORT CLICK**

**SLEEP MODE
= BLINKING WHITE**

2 SHORT CLICKS on
the rear button to
go back in SLEEP
mode

CHANGE THE MODE

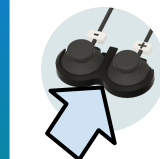

1 LONG CLICK on both
buttons of the control
pad

OR



1 SHORT CLIC on the
rear button
Short vibration = **OK**

FREE MODE = GREEN

Accompanies the movements

—
-100g /clic



+
+100g /clic

SETTING AN IDEAL COMPENSATION FORCE



1 LONG CLICK on the rear button
Short vibration = **OK**

PARKING POSITION ON WHEELCHAIR

In **FIXED** mode, pull down the ORTHOPUS Supporter on the armrest: the device stays blocked in place.



Rear button
blinking blue

Parking position activated = OK

To go out of the Parking position, click on the **+** button of the control pad.

WAITING FREE MODE



Rear button
blinking green

If the device is switched in FREE mode without arm in the arm brace, the ORTHOPUS Supporter puts itself in safety.

ERROR MODE



Unplug the device and wait a few minutes before using it again. If the Rear button stays red, please contact your ORTHOPUS Supporter Country Representative.

RECOMMENDATIONS

i While starting the ORTHOPUS Supporter, don't touch the control pad and don't install an arm in the arm brace.

i Prior to removing the user's arm from the support, please **always ensure** that the ORTHOPUS Supporter is in STATIONARY or SLEEP mode.

i It is recommended that users **become acclimated to the device gradually** and be monitored by a qualified healthcare professional.

i Only an **individual trained specifically** is authorized to install the ORTHOPUS Supporter.

To take full advantage of all that the ORTHOPUS Supporter has to offer, **please make sure:**

✓ In horizontal position, the arm isn't leaning forwards or backwards.

✓ The arm is always in contact with the elbow support. If necessary, set an upper swing limit.

✓ On the wheelchair, cables and box interface are protected for the rain.

IMPORTANT INFORMATIONS

! You must read the full user manual for the device orthopus.com/documentation

⊖ Do not apply excessive force to the device. For example, do not lean on it. **The ORTHOPUS Supporter is designed solely to support the arm.**

⊖ If there are any defects or damage to the electronic components, do not use the device. **Contact your medical equipment provider for any maintenance issues.**

⊖ **Do not modify any part of this equipment**, as this may result in the loss of the warranty.

! The device is resistant to water splashes but is **not waterproof**. Do not expose it to heavy rain, high humidity, or snowfall.



If the ORTHOPUS Supporter isn't mounted, **it must remain in its packaging and box at all times**. The box is to be kept should the device need to be returned.



ORTHOPUS Supporter Quickchart
v1.2 - 2025/03/04